

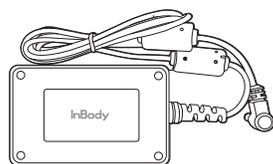



Your InBody 270 package comes with the following:

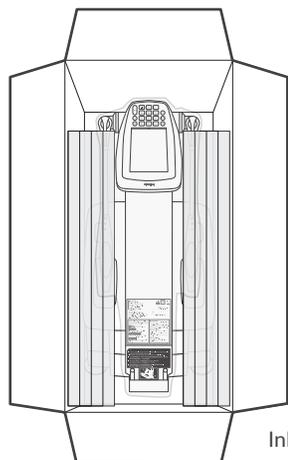


Adapter

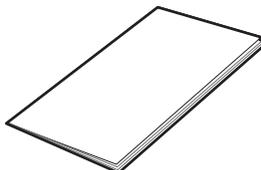

User's Manual

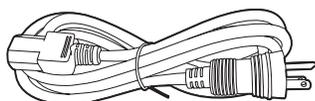


Power Cord

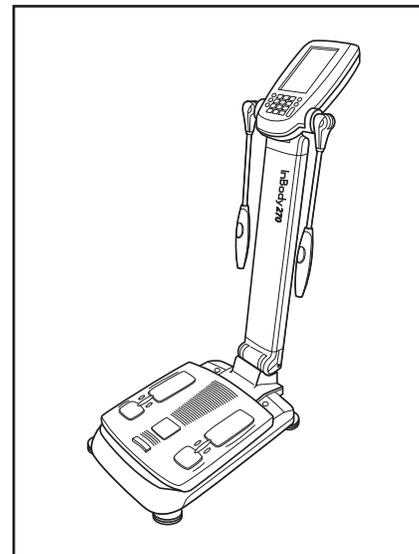

Do not cut too deep when opening up the unit box. Cutting too deep may damage the InBody unit and/or unit padding.


## UNBOXING + ASSEMBLY

1

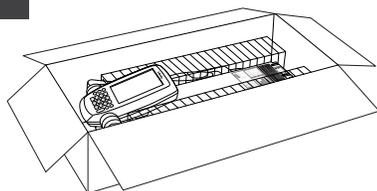


Keep the box right side up. Slice between the flaps to open the unit box. Do not cut too deeply or you may damage the contents. Unfold and take out all the padding on top to reveal the unit.

2

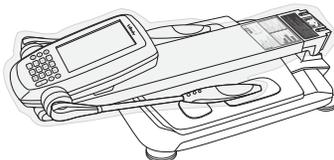


Lift the unit up from the box, making sure you are holding onto the footplate with both hands. Keep the footplate horizontal to the ground. *(This may require two people)*

3

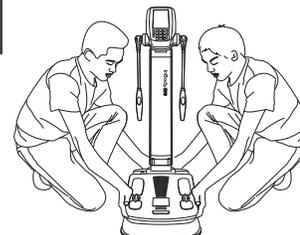


Once you place the unit on the ground, use one hand to hold the footplate and the other hand to push the head upright to a vertical position until you hear a click.

4

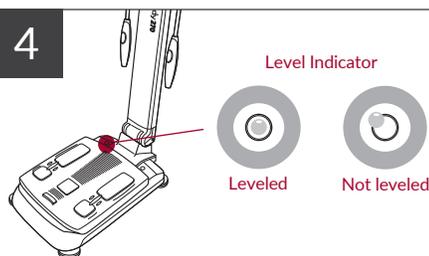


Tilt the head upright and straighten out the handles. Check the level indicator to see if the unit is balanced.

5

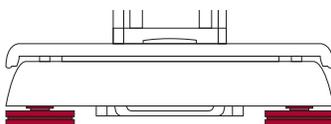


If not level, adjust the four footplate supports until the bubble is inside the red ring. Rotate clockwise to lower the height of the support or counterclockwise to raise it.

6

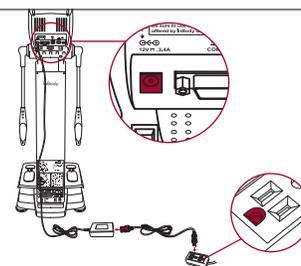

Connect the adapter to the power input port located on the rear panel of the unit. Connect the adapter to the power cord. Plug the power cord into an outlet.

## PASSCODE SETUP

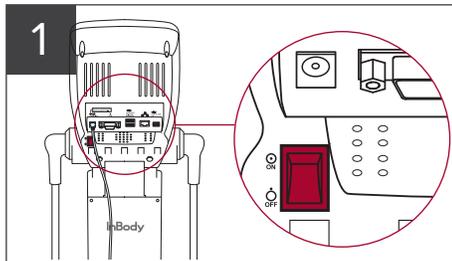

1

Turn on the InBody unit by using the switch on the back of the unit.

*(Note: Make sure there is nothing on top of the footplate when turning on the unit for proper calibration.)*

2



Once the home screen appears, access the "Administrator Menu" by using the default password "0000".


3



You'll be prompted to enter a new four-digit passcode. Continue to use "0000" if you do not wish to enter a new password.

*(Note: Due to HIPAA regulations, resetting a forgotten passcode will require signed documentation to be submitted to InBody by the owner or facility manager before reset steps are provided)*

## PRINTER SETUP

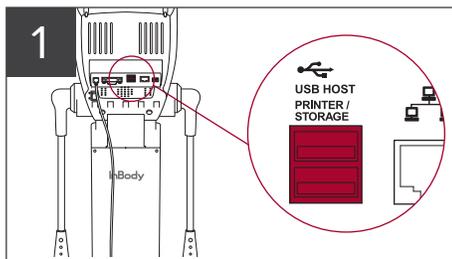

1


Insert the power cord into the printer's power slot and the other end into an outlet. Insert the USB cable into the printer's USB slot and the other end into the back of the InBody unit.

2



Load the InBody Result Sheets into the printer and press the power button to turn on.

1

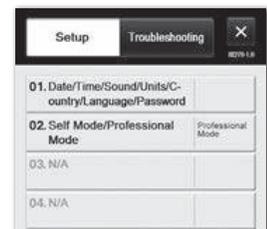

Select "option 01" inside the "Administrator Menu". Adjust to your current date and time and select "Save".

*(Time is displayed as a 24-hour format, but will show up on your Result Sheet as a standard 12-hour format.)*

## UPDATE YOUR LOGO



1

Locate your InBody USB thumb drive from the unit box. If requested, a pre-loaded logo file will be saved to your USB thumb drive.


*(If you did not request a modified logo and would like to receive one, please contact our support department at support@inbody.com with an image file of your desired logo and the InBody model type it will be used for.)*

2



Insert your USB with logo file inside into the unit while it's turned off. Turn on the device and select "option 14" inside the "Administrator Menu". Confirm the "Image Box" is checked off. If successful, your personalized logo image will be displayed in the "Preview of Custom Logo" area.

3

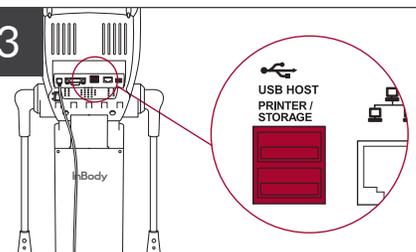

Remove the USB thumb drive from the unit and delete the "logo" image file using a computer to prevent any accidental re-upload of the logo files.

Exit the Administrator Menu for your first InBody Test.